

On securing the wound, prepare Medisorb A and a secondary dressing to cover the Medisorb A e.g. Medisorb F or Medisorb P. Choose the size of the secondary dressing that will cover the entire wound.

1




Before applying the dressing put on gloves and wash the wound with a 0.9% saline solution and dry the surrounding skin.

2



Select the appropriate dressing, covering the entire wound surface with an additional 2 cm margin.

3

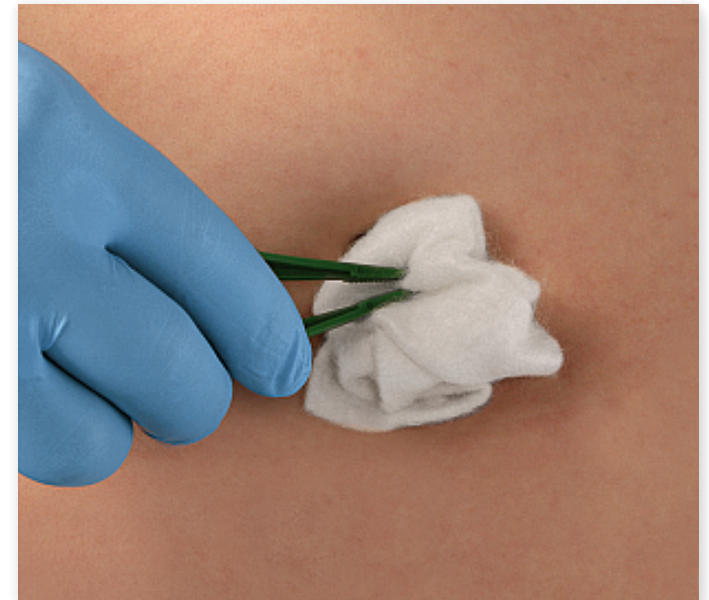

Put Medisorb A directly on the wound or into the wound. Then cover the wound with a secondary dressing e.g. Medisorb F which is transparent and allows the observation of the wound.

4




Remove the protective (white) paper from the central lower part of the dressing and holding by the paper edges apply the dressing on the skin.

5




Remove the protective (white) papers from the edges and stick the rest of the dressing. Lightly press the edges of the dressing.

6

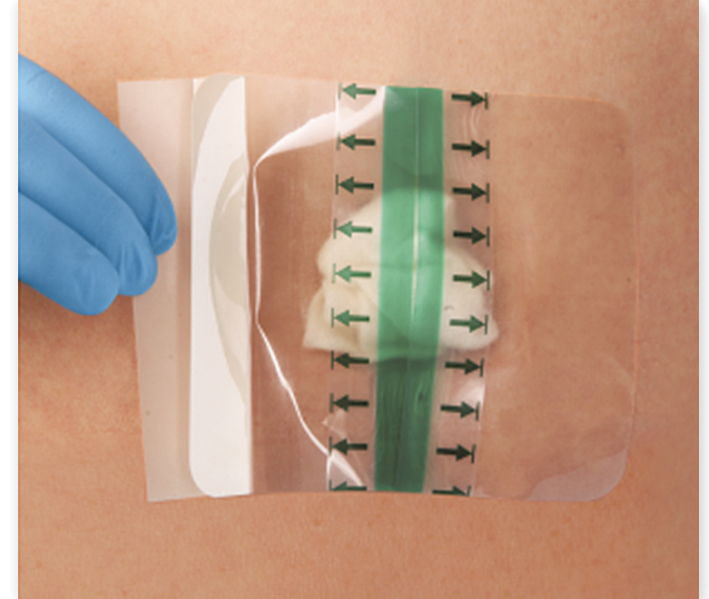


7

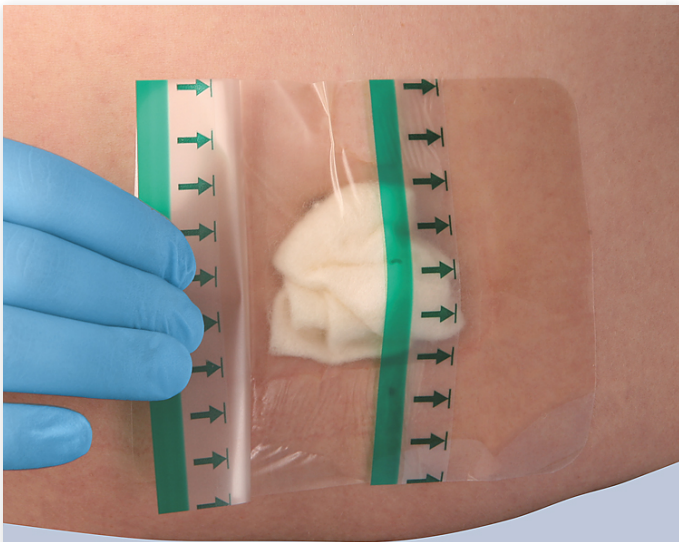

Remove the top plastic cover by pulling the tab in the direction shown by the green arrows.

8

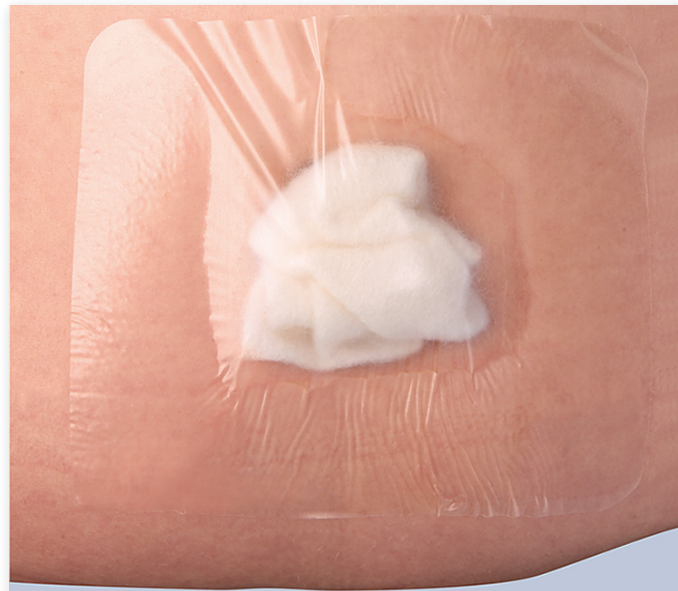

In the case of leg ulcers Medisorb A may be used in combination with compression therapy, if this is a medical recommendation. In uninfected wounds the dressing may stay for up to 7 days.

9



In uninfected wounds the dressing may stay for up to 7 days.

10



In infected wounds it may be used only under medical supervision.

11



The dressing change should occur when the surface of Medisorb A has entirely transformed into a gel. The interval between dressing changes depends entirely on the state of the wound.

12



Medisorb can easily and painlessly removed without disturbing the wound healing process: first remove the external dressing, and then using sterile forceps remove Medisorb A from the wound, and finally clean the wound of the residual gel washing the wound with a sterile 0.9 % saline solution.