

On securing the wound you should choose the size of the secondary dressing that will cover the entire wound with a 2-3 cm margin.

1



Before applying the dressing put on gloves and wash the wound with a 0.9% saline solution and dry the surrounding skin.

2



Select the appropriate size of the Medisorb P dressing so that it covers the wound surface, and 2-3 cm of the skin around it.

3



Remove the bigger protective paper and apply the dressing onto the wound.

4



Remove the second protective paper and stick the dressing gently pressing its edges.

5




In the case difficult to secure places (heel or elbow) to facilitate the adhering of the dressing, its sides can be cut according to individual needs.

6



Securing decubitus wounds in the sacral area, bend the dressing, put it in the buttocks hollow and gently pressing smooth it out. If the patient does not hold urine or faeces, the edges of the dressing must also protected with adhesive tapes.

7

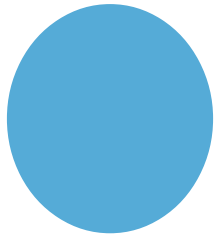

The dressing should be monitored daily to prevent any leakage of secretions. If there are no worrisome effects, the dressing can be left on the wound for up to 7 days. The dressing must be removed immediately after any signs of infection. The treatment may be continued after the elimination of inflammation.